Procare Elite

User Manual

Be well, Live well
Wellell

CONTENTS

ALTERNATING PRESSURE REDISTRIBUTION SYSTEM

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PLEASE READ THE MANUAL BEFORE USE

IMPORTANT SAFEGUARDS

READ ALL INSTRUCTIONS BEFORE USING

DANGER - To reduce the risk of electrocution:

- (1) Always unplug this product immediately after using.
- (2) Do not use while bathing.
- (3) Do not place or store this product where it can fall or be pulled into a tub or sink.
- (4) Do not place in or drop into water or other liquid.
- (5) Do not reach for a product that has fallen into water. Unplug immediately.

WARNING - To reduce the risk of burns, electrocution, fire, or injury to persons:

- (1) Evaluate patients for entrapment risk according to protocol and monitor patients appropriately.
- (2) The product may be used for patients with spinal injury, but suggested to consult with physician before use. However, it should not be used for patients with unstable spinal fractures.
- (3) Close supervision is necessary when this product is used on or near children. Electrical burns or choking accident may result from a child swallowing a small part detached from the device.
- (4) Use this product only for its intended use as described in this manual. Do not use other mattresses not recommended by the manufacturer.
- (5) Do not operate this product if it seems to be damaged or has been dropped into water. Return the product to the point of purchase for examination and repair.
- (6) Keep the cord away from heated surfaces.
- (7) Never block any air openings of this product or place it on soft surfaces, such as a bed or couch, where openings may be blocked. Keep the air opening free of lint, hair, and other similar particles.
- (8) Never drop or insert any object into any opening or hose.
- (9) Do not modify this equipment without authorization of the manufacturer.
- (10) Mattress covers have passed skin sensitization and skin irritation test. However, If you suspect that you may have had or are having an allergic reaction, please consult a physician immediately.
- (11) Do not leave long lengths of tubing around the top of your bed. It could lead to strangulation.
- (12) Third conductor in the POWER SUPPLY CORD is only a functional earth.

CAUTION: If there is a possibility of electro-magnetic interference with mobile phones, please increase the distance (3.3m) between devices or turn off the mobile phone.

NOTE, CAUTION AND WARNING STATEMENTS:

NOTE: Indicate some tips.

CAUTION: Indicate correct operating or maintenance procedures in order to prevent damage to or destruction of the equipment or other property

WARNING: Calls attention to a potential danger that requires correct procedures or practices in order to prevent personal injury.

SYMBOLS

	Catalog, reorder or reference number
	Manufacturer
	Complies with standards protecting against electric shock for type BF equipment.
	Consult operating instructions for use
	Attention, should read the instructions.
	Attention, should read the instructions.
IP21 	Protected against solid foreign objects of 12.5 mm and greater; Protection against vertically falling water drops
	Class II
	Temperature limitation/temperature range
	Stand-by
	Dry clean, Any Solvent Except Trichloroethylene
	Do Not Iron
	Tumble Dry, Normal, Low Heat
	Do Not Tumble Dry
	Do Not Bleach
	Do Not Dry Clean
	Machine wash, regular / normal, 60 degrees C (140 degrees F)
	Attention – Observe proper Disposal of Electrical & Electronic Equipment (WEEE): This product should be handed over to an appropriate collection point for the recycling of electrical and electronic equipment. For more detailed information about the recycling of this product, please contact your local city office, household waste disposal service or the retail store where you purchased this product.

1. Introduction

This manual should be used for initial set up of the system and for reference purposes.

1.1. General Information

The system is a high quality and affordable mattress system suitable for treatment and prevention of pressure ulcers.

The system has been tested and successfully approved to the following standards:

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EMC Warning Statement

This equipment has been tested and found to comply with the limits for medical devices to the EN 60601-1-2. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving device.
- Increase the separation between the equipment.
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) are connected.
- Consult the manufacturer or field service technician for help.

1.2. Intended Use

This product is intended to help and reduce the incidence of pressure ulcers while optimizing patient comfort. It also provides following purposes:

- to help and reduce the incidence of pressure ulcers while optimizing patient comfort.
- for long term of patients suffering from pressure ulcers.
- for pain management as prescribed by a physician.

The product can only be operated by personnel who are qualified to perform general nursing procedures and have received adequate training in knowledge of prevention and treatment of pressure ulcer.


 **NOTE: Equipments are not suitable for use in the presence of flammable anesthetic mixture with air or with oxygen or nitrous oxide.**

2. Product Description


Unpack the box to check for any damage which may have occurred during shipment. If there are damages, please contact the point of purchase immediately.

2.1. Pump and Mattress System

- (1) CPR
- (2) Pump unit
- (3) Mattress System
- (4) Quick Connector

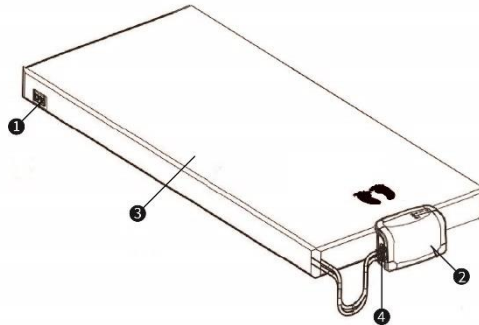
- (1) CPR
- (2) Pump unit
- (3) Mattress System
- (4) Quick Connector
- (5) Side Bolster (option)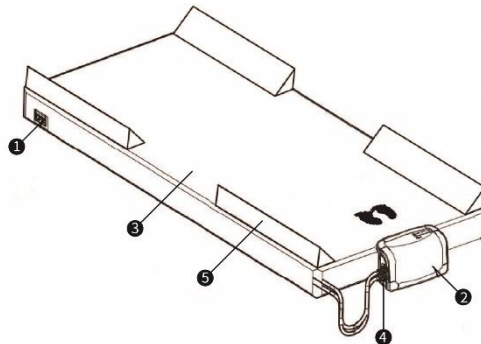

2.2. Pump Unit Front

- (1) Quick Connector
- (2) Front Panel
- (3) Air Filter

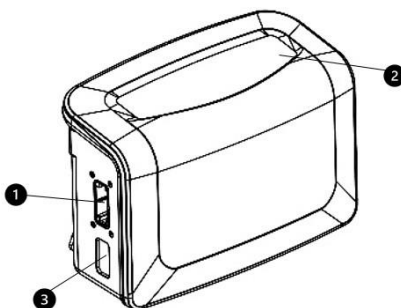
Side

- (1) Power Switch
- (2) Power Cord
- (3) Fuses


Rear

- (1) Air filter
- (2) Mounting brackets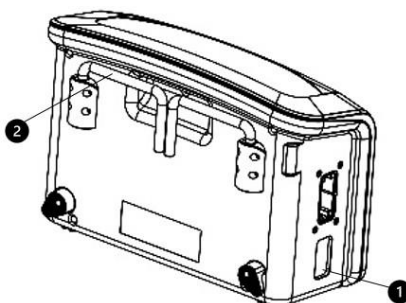

2.3. Front Panel

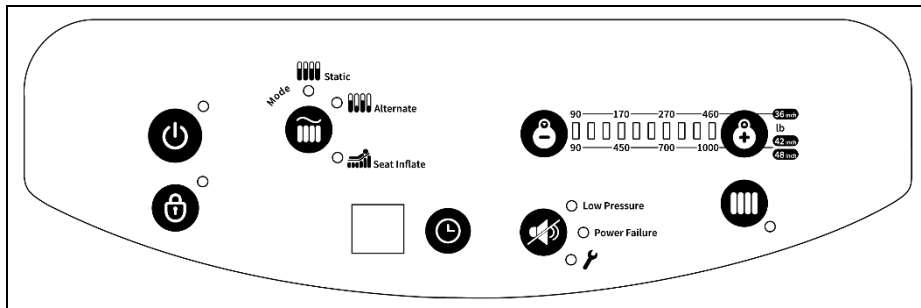(1) Pressure Adjust Levels

This controls the air pressure output. When the firmness (+) is increased, the output pressure will increase and vice versa for decreasing (-) air pressure. When a patient's condition is significantly changed, please reassess the setting level.

NOTE: The surface will go into Max firm automatically when the power switch is turned on. Once the max pressure level is reached, the pump will automatically switch into alternating mode.

NOTE: Check if the pressure is suitable for the patient by sliding one hand beneath the air cells at the level of the patient's buttocks. Always leave at least 1" inch space between patient and the static cell to prevent bottoming out.

(2) Cycle Time

There are four selectable cycle times found from the touch panel area. By pressing the button, user can select one of four cycle times based upon patient comfort and desired outcome.

(3) Modes**A. Max Firm mode**

The surface will go into Max firm automatically when the power switch is turned on. This insures the pump is able to reach its maximum operating pressure. Once the max pressure level is reached, the pump will automatically switch into alternating mode. User can also use this function during patient transfers or nursing procedures for better support.

NOTE: Max Firm and preset function LED will be illuminated simultaneously until the mattress is reached to its maximum pressure.

B. Alternate Mode

Alternating air cells are partially deflated and inflated to avoid prolonged pressure on any single point beneath the patient.

C. Static Mode

Non-alternating mode, all of the air cells are equally inflated at lower pressures when compared to the respective comfort level in alternating mode.

D. Seat Inflation mode

The seat inflation features additional supports to the patient during upright position without bottoming out. User can select this additional feature under either static or alternate mode.

(4) Panel Lock

If the control panel remains untouched for 5 minutes, a green LED will light and the control panel will lock all settings. This will prevent the settings from being accidentally changed during normal operation. To unlock, simply press the Panel lock button for **two** seconds.

(5) Alarm Mute

When the Alarm Mute button is pressed, the buzzer will be temporarily suspended. If the situation is not resolved within 3 minutes, the alarm will activate.

A. PFA (Power Failure Alarm)

During a power failure situation, the Power Failure LED light and buzzer will activate. To disable both buzzer and LED, press the mute button.

B. Low Pressure Indicator

When low pressure LED lights up, the pressure inside of air mattress is below normal. Please refer to troubleshooting.


C. Tech. Support

The Service indicator light will activate when an issue occurs with the system. User can notify the technician for repair.

3. Installation

3.1. Pump & Mattress Installation

⚠ NOTE: Unpack the box to inspect for any damage which may have occurred during shipment. If there are any damages, please contact the point of purchase immediately.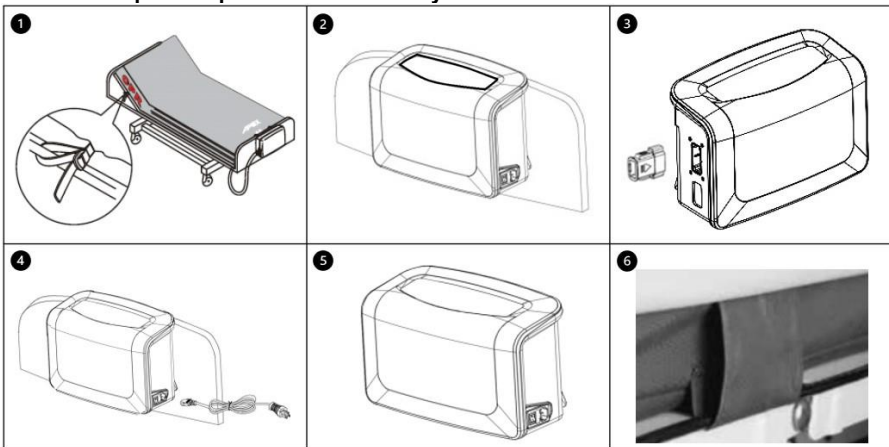

- (1) Place the mattress or pad on top of the bed frame. Please note for the foot end. There are securing straps on the base of the mattress. Secure the mattress firmly by fixing the straps to the bed frame, ensure that moving sections of the bed frame are still free to move.

⚠ WARNING: The overlay mattress must be applied on the underlying mattress

- (2) Hang the pump onto bed rail (foot-end), and adjust hangers to best upright position of the pump, or place the pump on a flat surface.
- (3) Connect air hose connectors from air mattress to the pump unit. When a "click" sound is felt or heard, the connection is completed and secured.

⚠ NOTE: Check and ensure the air hoses are not kinked or tucked under mattress.

- (4) Plug the power cord into electrical outlet.

⚠ NOTE:

- Make sure the pump unit is suitable for the local power voltage.
- The plug is also served to disconnect the device.

- (5) Turn the power switch to ON position.

⚠ CAUTION: The pump can only be applied to the mattress recommended by the manufacturer. Do not use it for any other purpose. (Applied part: air mattress)

- (6) The cable management loops are provided along the sides of the mattress to cover the power cable. Please make sure the cable has no signs of damage. After installation, the extra length of the power cord, if any, should be neatly managed to avoid causing a trip and should be clear of moving bed mechanisms or other possible entrapment area.

The EQUIPMENT should be firmly placed at position where users/doctors can access easily.

4. Operation


 **NOTE: Always read the operating instruction before use.**

4.1. General Operation

- (1) Turn on the power switch to run.
- (2) Allow 20~25 minutes for the mattress to inflate fully.
- (3) When the mattress sets up, it will execute the initial inflation (Maxifirm). The indicator of Maxifirm (LED) will constantly flash until the target pressure is reached, then the system will automatically enter the pre-set mode.
- (4) According to the weight and height of the patient, adjust the pressure setting to the most comfortable level without bottoming out. After the pressure in mattress reach the intended level, the air mattress is ready to use.

4.2. Emergency CPR Operations

 **NOTE: When there is an emergency to perform CPR on the patient, quickly turn the CPR knob to release air from mattress. The CPR knob is located at the head-end, right-hand-side of the mattress. The quick connector on the pump unit can be disconnected for even faster deflation.**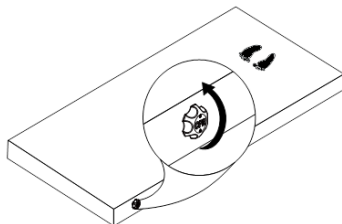


 **NOTE: There will be 2 CPR knobs on the bariatric mattress (42" & 48").**

4.3. Transportation

 **NOTE:** When there is a power outage or a need of transporting the patient, one can capped on the quick connector as shown on the left to slow the air leakage. Depending on the setting and mattress height, the mattress can remain inflated for a while.

 **WARNING:** If there is a bottom-out issue, please remove the patient from deflating mattress to another proper surface.

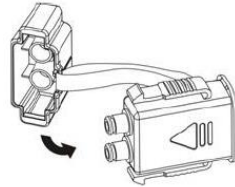4.4. Heel Relief Function

The last five cells from the foot-end have heel-relief connectors. Carers can easily disconnect the connector and heels can be suspended in air to completely eliminate pressure exerted on the heels.

For permanent off-loading/pressure relief:

- Select **no more than two** cells directly under the area that desired to relieve.

 **NOTE:** This one to two cells can be left permanently deflated.

 **NOTE:** Deflating more than two adjacent cells may affect the support of the patient during the normal alternating cycle.

5. Cleaning

It is important to follow the cleaning procedures to avoid cross contamination. Be sure to clean the surface in a dry and dust free environment. Wipe down the pump unit with a damp cloth pre-soaked with a mild detergent. Avoid contact with dust and proximity to dusty areas. Make sure that any cleaning agents you use will not harm or corrode the plastic casing on the pump unit. If your doctor or medical facilities have other special cleaning instruction, please follow the professional instruction.

CAUTION: Do not immerse or soak pump unit.

WARNING: Do not remove the housing of the pump to avoid the electrical shock. All disassembly or repair should be done by professional technicians.

CAUTION: The pump does not need oil lubrication; please do not dissemble the system.

Cover Material: Nylon/PU	
---	--

Wipe-down the mattress unit with a damp cloth pre-soaked with warm water containing a mild detergent, or chlorine bleach followed by an approved intermediate level disinfectant. Also the mattress top cover can be completely removed for laundry with water temperature indicated on the laundry label; however, it is recommended that the user still check with local policy to determine the time/ temperature ratio required achieving thermal disinfection. The cover may also be cleaned using sodium hypochlorite diluted in water. After cleaning, please avoid dust and proximity to dusty areas and all parts should be air dried thoroughly before use.

CAUTION: Do not use phenolic based products for cleaning.

CAUTION: After cleaning, before cleaning process starts, remove the four foams inside the pockets when the optional cover with side bolster is used. Dry the mattress without direct exposure of sunlight.

6. Storage

- (1) To store the mattress, lay the mattress out flat and upsides down.
- (2) Roll from the head end towards the foot end with CPR valve open.
- (3) Foot-end strap can then be stretched around the rolled mattress to prevent unrolling.

NOTE: Do not kink, crease or stack the mattresses and do not store the system in direct sunlight, high temperature or moisture area.


7. Maintenance

7.1. General

- (1) Check main power cord and plug if there is abrasions or excessive wear.
- (2) Check mattress cover for signs of wear or damage. Ensure mattress cover and tubes are stubbed together correctly.
- (3) Check airflow from the quick connector. The airflow should alternate between each connector every half-cycle time if it's in alternating mode.
- (4) Check the air hoses if there is kink or breaks. For replacement, please contact the point of purchase.

7.2. Fuse Replacement

- (1) Disconnect the plug from power supply when a blown fuse is suspected.
- (2) Use a proper tool to remove the cover of the fuse holder.
- (3) Insert a new fuse of the correct rating in, and replace the fuse holder back to correct position. The fuse should be rated as F1AL /250V type.

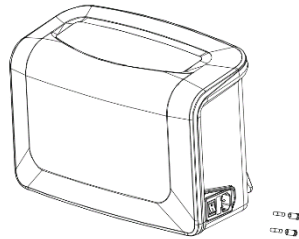7.3. Air Filter Replacement

- (1) Replace the air filter located at the side of pump.
- (2) The filter is reusable and can be washed gently with a mild detergent and water. Air dry the filter before use.
- (3) Check and replace air filter regularly.

8. Expected Service Life

The products are intended to offer safe and reliable operation when use or installed according to the instructions provided by Wellell. Wellell recommends that the system be inspected and serviced by authorized technicians if there are any signs of wear or concerns with device function and indication on products. Otherwise, service and inspection of the devices generally should not be required.

9. Trouble Shooting

Q1. Power is not ON

- Check if the power plug is connected.
- Check for a blown fuse.

Q2. Low Pressure Alarm is on

- Check if the Quick Connector is tightly secured.
- Check if all tubing connections along mattress are secured.
- Check if the air hoses are kinked or broken.

Q3. Power Failure Alarm is on

- Check if the power is suddenly shut down.
- Check if the power cord is connected properly.

Q4. Patient is bottoming out

- Pressure setting might be inadequate for the patient, adjust comfort range 1 to 2 levels higher and wait for a few minutes for best comfort.

Q5. Mattress form is loose

- Check if all the snap buttons or straps of mattress are all securely fastened.
- Check if the mattress is fixed to the bed frame by straps.

Q6. No air produced from some air outlets of the air tube connector

- This is normal since there is alternating mode. Air outlets take turns to produce air during their cycle time.

If the above information does not solve your problems, please contact the point of purchase directly. They might require a technician to take care the problem.

10. Technical Specification:

Item		Specification	
Power Supply (Note: See rating label on the product)		AC 110-120V 60 Hz, 0.17A	
Fuse Rating		F1AL, 250V	
Cycle time		Four selectable (10,15,20,25mins)	
Dimension (L x W x H)		34.0 x 15.0 x 23.4 cm / 13.4" x 5.9" x 9.2"	
Weight		3.1 Kg / 6.83 lb	
Environment	Atmospheric Pressure	700 hPa to 1060 hPa	
	Temperature	Operation: 10°C to 40°C (50°F to 104°F) Storage: -15°C to 50°C (5°F to 122°F) Shipping: -15°C to 70°C (5°F to 158°F)	
	Humidity	Operation: 10% to 90% non-condensing Storage: 10% to 90% non-condensing Shipping: 10 % to 90% non-condensing	
Classification		Class II With Function Earth, Type BF, IP21 Applied Part: Air Mattress  Not suitable for use in the presence of a flammable anesthetic mixture (No AP or APG protection)	
Mattress		Specification	
Model	8" Mattress 2" foam pocket is provided		
sizes of mattress	36" mattress	42" mattress	48" mattress
Dimension (L x W x H)	200 x 90 x 20.3cm 78.7" x 35.4" x 8"	200 x 107 x 20.3 cm 78.7" x 42" x 8"	200 x 122 x 20.3 cm 78.7" x 48" x 8"
Weight	7.5 kg / 16.5 lbs	8.5 kg / 18.8 lbs	9.5 kg / 21.0 lbs
Maximum Weight Capacity	210 kg/ 460 lbs	455 kgs 1000 lbs	455 kg 1000 lbs

 NOTE:

- Consult the distributor or EU representative for further technical documents.
- The specification is also suitable for other areas operating with same power supply.
- Mattress dimension and weight is measured without foam cushion
- The manufacturer reserves the right to modify the specification without notice.

Appendix A: EMC Information

Guidance and Manufacturer’s Declaration- Electromagnetic Emissions:

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

Emissions Test	Compliance	Electromagnetic Environment-Guidance
RF emissions CISPR 11	Group1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network
Harmonic emissions IEC61000-3-2	Class A	
Voltage fluctuations / Flicker emissions IEC61000-3-3	Complies	
 Warning: 1. The device should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the device should be observed to verify normal operation in the configuration in which it will be used. 2. Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation. 3. Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Pump, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.		

Guidance and Manufacturer's Declaration- Electromagnetic Immunity:

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

Immunity Test Levels				
Basic EMC standard	Professional healthcare facility environment	HOME HEALTHCARE ENVIRONMENT	Compliance Levels	Electromagnetic Environment-Guidance
Electrostatic Discharge (ESD) IEC61000-4-2	±8kV contact ±15kV air		±8kV contact ±15kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/ burst IEC61000-4-4	±2kV for power supply line ±1kV for input/output line		±2kV for power supply line ±1kV for input/output line	Mains power quality should be that of atypical commercial or hospital environment
Surge IEC61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s)	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	Mains power quality should be that of atypical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC61000-4-11	Voltage Dips: i) 100% reduction for 0.5 period, ii) 100% reduction for 1 period, iii) 30% reduction for 25/30 period, Voltage Interruptions: 100% reduction for 250/300 period		120V	Mains power quality should be that of a typical commercial or hospital environment. If the user of this device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC61000-4-8	30 A/m	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Conducted RF IEC 61000-4-6	3 Vrms 0,15 MHz – 80 MHz 6 Vrms in ISM bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	3 Vrms 0,15 MHz – 80 MHz 6 Vrms in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	6Vrms	Portable and mobile RF communications equipment should be used no closer to any part of this device, including cables, than there commended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \sqrt{P}$ 150kHz to 80MHz $d = 0.6\sqrt{P}$ 80MHz to 800MHz $d = 1.2\sqrt{P}$ 800 MHz to 2.7GHz
Radiated RF EM Fields IEC61000-4-3	3 V/m 80 MHz to 2.7 GHz 80 % AM at 1 kHz 385-6000 MHz, 9-28V/m, 80% AM(1kHz) pulse mode	10 V/m 80 MHz to 2,7 GHz 80 % AM at 1 kHz 385-6000 MHz, 9-28V/m, 80% AM(1kHz) pulse mode and other modulation	3V/m	Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). ^b Field strengths from fixed RF transmitters, as determined by an

	and other modulation			electromagnetic site survey ,^a should be less than the compliance level in each frequency ranged. Interference may occur in the vicinity of equipment marked with the following symbol: 
NOTE 1: U_T is the a.c. mains voltage prior to the application of the test level NOTE 2: At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 3: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people				
a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device. b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.				

Recommended separation distances between portable and mobile RF communications equipment and this device:

This device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of this device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and this device as recommended below, according to the maximum output power of the communications equipment

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = \sqrt{P}$	80 MHz to 800 MHz $d = 0.6\sqrt{P}$	800 MHz to 2.7 GHz $d = 1.2\sqrt{P}$
0.01	0.1	0.06	0.12
0.1	0.31	0.19	0.38
1	1	0.6	1.2
10	3.1	1.9	3.8
100	10	6	12
For transmitters rated at a maximum output power not listed above, the recommended separation distance <i>d</i> in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.			

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All rights reserved
Wellell America Corp.
**Wellell Inc.**